



2 in 1 cleaning & disinfection



Product description

Highly effective formulation of quaternary ammonium compounds, polymeric biguanide and surfactants, working in synergy to deliver a potent cleaning and disinfection solution.

1

Features & benefits

Cleans and disinfects in one go; kind to surfaces and skin friendly; residual efficacy - antimicrobial properties present for up to 24 hours after application; AZOMAX™ is 99.99999%* effective against pathogens validated by European Standard EN 16615.

2

Surface

Multi-surfaces, suitable for surfaces and equipment sensitive to alcohol.

3

Antimicrobial efficacy

Test	Test organism	Contact time	Test conditions	Log reduction
EN 16615	<i>Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Candida albicans</i>	5 min	Dirty	>7 LOG
EN 1276	<i>Proteus mirabilis</i>	1 min	Clean	>5 LOG
EN 1276	<i>Acinetobacter baumannii</i>	1 min	Clean	>5 LOG
EN 14476	H1N1	5 min	Dirty	>3 LOG
EN 14476	Norovirus	5 min	Dirty	>4 LOG
EN 14476	BVDV	5 min	Dirty	>4 LOG
EN 1276	<i>Salmonella enterica</i>	5 min	Dirty	>5 LOG
EN 1276	<i>Klebsiella pneumonia</i>	5 min	Dirty	>5 LOG
EN 13727	<i>E.coli</i>	5 min	Dirty	>5 LOG
EN 13727	<i>Listeria</i>	5 min	Dirty	>5 LOG
EN 13727	<i>Legionella</i>	5 min	Dirty	>5 LOG
EN 13727	<i>Staphylococcus aureus</i>	5 min	Dirty	>5 LOG
EN 13727	MRSA	5 min	Dirty	>5 LOG
EN 13727	<i>Pseudomonas aeruginosa</i>	5 min	Dirty	>5 LOG
EN 13727	<i>Enterococcus hirae</i>	5 min	Dirty	>5 LOG
EN 13727	VRE	5 min	Dirty	>5 LOG
EN 1040	<i>Pseudomonas aeruginosa</i>	5 min	Dirty	>5 LOG
EN 1040	<i>Staphylococcus aureus</i>	5 min	Dirty	>5 LOG
EN 13624	<i>Candida albicans</i>	5 min	Dirty	>4 LOG
EN 14476	Vaccinia	60 min	Dirty	>4 LOG
EN 14348	<i>Mycobacterium terrae TB</i>	60 min	Dirty	>5 LOG
EN 14476	MERS Coronavirus	5 min	Dirty	>4 LOG

4



Use AZOMAX™ wipe for cleaning and disinfection. Use more than one wipe if necessary so the surface stays visibly wet, then leave to dry.



Once used, discard the wipe into appropriate waste container.



Use AZOMAX™ spray to clean – spray solution onto contaminated surface and wipe with clean cloth. To disinfect – spray onto the cleaned surface and wipe or leave to dry. To deodorise – spray onto contaminated surface and leave to dry.

Ensure the surface or a piece of equipment is clean prior to disinfection. Do not use one wipe on more than one piece of equipment. Do not mix with any other chemicals. Keep the canister/spray bottle/bucket closed. Store in a cool, dry place. Avoid direct contact with sunlight. Use biocides safely. Always read the label and product information before use.

Hospital wards



Mattresses, bed frames, commodes, table tops, chairs, wheelchairs, hoists, trolleys, door handles, light switches, non-invasive class IIa medical devices

NPSA - The Revised Healthcare Cleaning Manual 2009

Dental surgeries



Curing lamps, spittons, hand controls, local work surfaces, furniture & fittings, chairs, trolleys, door handles, light switches, non-invasive class IIa medical devices

HTM 01-05: Decontamination in primary care dental practices 2013

GP practises



Examination couches, table tops, chairs, trolleys, door handles, light switches, telephones, keyboards, non-invasive class IIa medical devices

NICE CG139 - Infection prevention and control of healthcare-associated infections in primary and community care 2012

Nursing homes



Mattresses, bed frames, table tops, chairs, wheelchairs, trolleys, door handles, light switches, other furniture & fittings, non-invasive class IIa medical devices

The Health and Social Care Act 2008 - Code of practice on the prevention and control of infections and related guidance

Code	Product	Description	Size (mm)	Wipes qty	Case qty
AM025W	AZOMAX™	Flow wrap	330x220	25	25
AM050W	AZOMAX™	Flow wrap	330x220	50	20
AM100WF	AZOMAX™	Flow wrap	295x220	100	15
AM100W	AZOMAX™	Canister	220x200	100	12
AM200W	AZOMAX™	Canister	180x200	200	12
AM300W	AZOMAX™	Bucket	260x235	300	4
AM500S	AZOMAX™ SPRAY	Bottle	500ml	N/A	12
AM050WCE	AZOMAX™	CE mark - flow wrap	300x220	50	30
AM100WCE	AZOMAX™	CE mark - canister	130x180	100	12
AM200WCE	AZOMAX™	CE mark - canister	180x200	200	12
AM500SCE	AZOMAX™ SPRAY	CE mark - bottle	500ml	N/A	12